INDICATIONS

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**Useful tips when first using the XL
The larger bore requires less pressure for injection.**

The excellent flexibility of the XL enables gentle insertion of the needle into the tissues.

INDICATIONS

-Threaded plastic hub.
-Needles can be used with all syringes and syringe systems in common use in dental anaesthesia.
The size of the needle is shown on the safety seal.

DIRECTIONS FOR USE

- 1.-Break the sterility guarantee seal just before use by turning the two parts of the protective sleeve in opposite directions.
- 2.-Completely separate the two parts of the protective sleeve by removing the sleeve's cap to reveal the end of the needle which will be fixed onto the syringe.
- 3.-Screw the plastic nozzle of the needle onto the threaded hub of the syringe by holding the second part of the sleeve which protects the other end of the needle.
- 4.-Remove the plastic protector with a slight twisting action just before the injection is administered. (It can be used again to remove the needle from the syringe to help avoid the risk of pricking).
- 5.-Unscrew the needle from the syringe carefully using the plastic protector or other means to help prevent risk of a needle prick.
Discard the needle into a waste disposal container in accordance with valid regulations.

WARNINGS

Extreme physical pressure or force may cause the needle to break.
Avoid extreme or repeated movement of the needle during injection as this may cause the needle to break.
DO NOT RECAP NEEDLES BY HAND.
Do not bend, break, or otherwise manipulate needles by hand.
Protect needles against high temperature and moisture.
Do not exceed the expiry date shown on the outer packaging.
The user is entirely responsible for any failure to comply with warnings
Store in dry and clean place